MICROKLEEN™ HS

High Suds Neutral Detergent

MicroKleen™ HS High Suds Neutral Detergent is specially formulated for the manual cleaning of glass, plastic, rubber and metal surgical instruments and equipment. It removes blood, tissue and soil, and leaves instruments moving freely and spot-free.

- **Rinses spot & residue-free.**
- **Safe and compatible with all metals.**
- **Reduces likelihood of rust, tarnish or corrosion.**

Product Features	Consumer Benefits
Rust Inhibitors:	Reduces likelihood of rust, tarnish or corrosion.
Rinsing Agents:	Rinses spot & residue-free.
Neutral pH:	Product safe and compatible with all metals.
Concentrated:	Low cost per use.
Bacteriostatic:	If stored properly, not favorable to bacterial growth.
Environmentally Friendly:	Biodegradable, phosphate-free, VOC free formula.

Procedures	Directions for Use
Directions:	Wear appropriate personal protective equipment based on safe work practices.
Manual Washing:	Use 1/8 fl. oz. (3.7 mL) to 2 fl. oz. (59 mL) per gallon (3.8 L) of warm water. Soak instruments 2-3 minutes or longer if required. Solution should be aspirated through lumens and channels of instruments. Scrub instruments under water and rinse thoroughly with warm water, aspirating water through channels to completely remove all traces of solution. Discard solution after each use, or when visibly soiled.
	Dilutions vary depending on temperature and water quality.
	NOT FOR USE IN AUTOMATIC WASHING EQUIPMENT.

Order No.	Size
B2HC	1 Gallon (3.8 Liters)

Physical Properties	
Form:	Liquid
Color:	Pink
Odor:	Detergent
pH Concentration:	6.9

Uses			
<input type="checkbox"/>	Presoak	<input type="checkbox"/>	Automatic Washer
<input checked="" type="checkbox"/>	Manual	<input type="checkbox"/>	Automatic Endoscope Reprocessor
<input type="checkbox"/>	Ultrasonic	<input type="checkbox"/>	Cart Washer

Temperature	
Usage:	68°-110° F (20°-43° C)
Storage:	Cool, dry area

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